Registered No. 2014 - 16561

# NOTARIAL CERTIFICATE

Chung Jin Law Group

Korean Re Bldg 5F #502, 68 Jong-ro 5-gil  
Jongno-gu, Seoul, Korea

## DECLARATION

J-Meditech Co.  
do hereby solemnly and sincerely declare

1. That the attached document(s) :

USER'S GUIDE

2. ☒ is (are) true and correct.

☐ is (are) true translation for the text(s) originally written in  
the Korean language conscientiously believing the same to  
be true and correct.

Signature

J-MEDITECH

*K. J. Kim*

President Ki-jo, kim

# USER'S GUIDE

Disinfecting unit, endoscope, flexible

J 1000

COPY OF ORIGINAL

JMEDITECH

# Table of contents

1. Introduction .....3

2. Purpose .....3

3. Overview of unit .....3

    Front view .....3

    Rear view .....3

    Interior view of Basin .....4

4. Installation .....5

5. Start up precautions .....5

6. Operation procedure .....5

7. During operation.....9

8. Storage & maintenance .....10

9. Technical specification .....10

10. Customer service .....11

11. Trouble shooting .....11

COPY OF ORIGINAL 76/11

## 1. INTRODUCTION


This product provides better quality maintenance of the endoscope to patients, providing them with an effective process for safe cleaning and sterilizing, thereby safeguarding against secondary infection through clinical endoscopy, using disinfectants.

## 2. PURPOSE

The medical equipment is able to sterilize by circulating the disinfecting fluid in the tube of a flexible endoscope.


## 3. OVERVIEW OF UNITS

Front view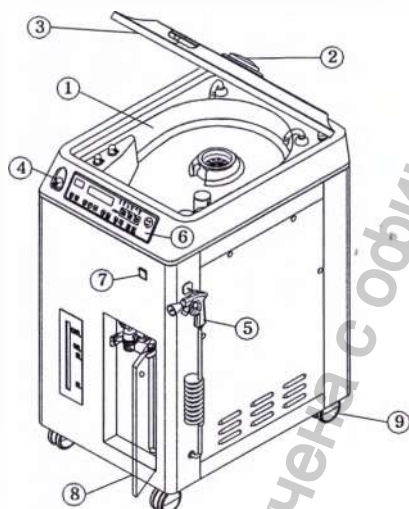

|   |                             |
|---|-----------------------------|
| ① | Basin                       |
| ② | Rotor                       |
| ③ | Basin lid                   |
| ④ | Connector for leakage check |
| ⑤ | Air gun                     |
| ⑥ | Control panel               |
| ⑦ | Power on/off switch         |
| ⑧ | Outlet (Drain) cover        |
| ⑨ | Moving wheel                |


Rear view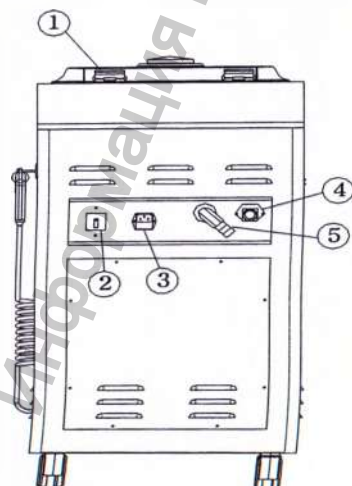

|   |                 |
|---|-----------------|
| ① | Hinge           |
| ② | Circuit breaker |
| ③ | Noise filter    |
| ④ | In-let          |
| ⑤ | Out-let (Drain) |

COPY OF ORIGINAL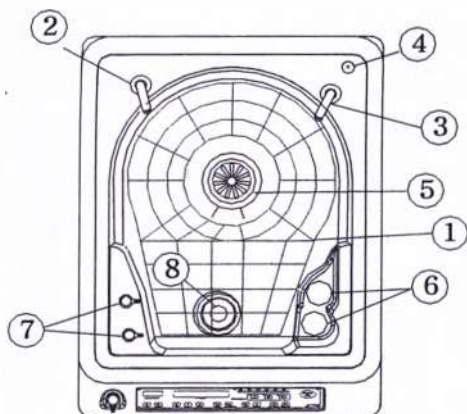

|   |                       |
|---|-----------------------|
| ① | Endoscope stand       |
| ② | Leakage check feed    |
| ③ | Disinfectant feed     |
| ④ | Gas Out-let           |
| ⑤ | Auxiliaries           |
| ⑥ | Water pressure sensor |
| ⑦ | Channel supplier      |
| ⑧ | Out-let (Drain)       |


#### 4. INSTALLATION

##### Requirements

- ① The water in/out-let should be located near to place where you install the machine.
- ② The water in/out-let should be drawn through a purifier system, in order to preserve the sterilizer and endoscope from rust and other contaminants.
- ③ The waste water drain hole should be located near the machine. (The water used in the sterilization process is expelled from the out-let (Drain) pipe during operation)

##### Installation procedure

- ① Connect the In-let hose to the In-let pipe using 15A ball-valve and connect the end of the ball-valve to the main in-let supplier.
- ② Seal the connecting point of the out-let hose and the out-let pipe after inserting it into the out-let drain hole.
- ③ The pipe should be connected to an electrical earth wire, to protect the user from electric shock by leakage, when the equipment is operating.

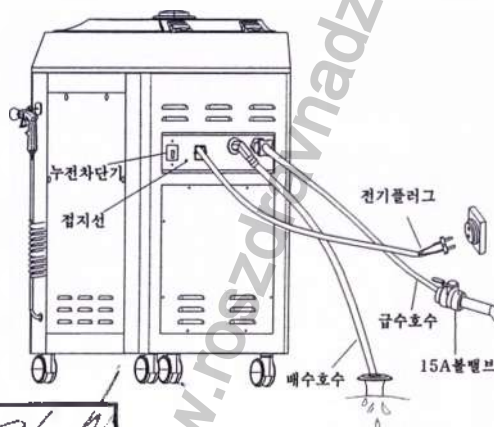COPY OF ORIGINAL

- ◆ A blockage in the in/out-let line can be a source of malfunction when the equipment is operating. The user should monitor the flow rate and contact the SVC center if some problem occurs regarding the flow rate in the in/out-let lines.

## 5. Start up precautions

- ① Check the power is stable as specified on user's guide (220V/60Hz.)
- ② Check the water on the in-let supply lines is rust free and uncontaminated.
- ③ Check the electrical earth wire is connected to the pipe.
- ④ Check the instrument is installed on a flat surface.
- ⑤ Check the connections of the In-let supply lines.
- ⑥ Check the connections of the Out-let (Drain) lines.

## 6. OPERATING PROCEDURE

### CONTROL PANEL

#### ① Cycle count

Displays the number of times the disinfectant has been used, and shows an Error code.

#### ② LCD Monitor display

Displays the operating status of each function, the setting and the time remaining.

#### ③ Progress display

Displays the progress status with lights, showing the running of the program.

#### ④ Selection of Program mode

Consists of three kinds of program mode, (A, B, C). Users can select any one of them (Cleaning, Sterilization and Air) after setting the time.

#### ⑤ Control of the disinfectant count

Provides the user with control of the count of the disinfectant usage, giving consideration to a particular disinfectant's characteristics.

#### ⑥ Time control

Lets the operator set the program mode, Cycle counter, the time of the leakage test and pressure, using the "Up", "Down" and "Enter" buttons.

#### ⑦ Leakage test

Allows the user to check the leakage source on the endoscope, selecting either the air pressure test or under water test.

#### ⑧ Air/Out-let

The "Air" button is used to drain the remained water in the endoscope channel, and the "Out-let" button is used to drain the remaining disinfectant in the basin.

### ⑨ In/Out-let of Disinfectant

The "In-let" button is used for supplying the disinfectant. Press the button "stop" when it is filled to a suitable level. The "Out-let" button is used to drain the disinfectant in the basin.

### ⑩ START/STOP

The "START/STOP" button is used when it is ready to operate. The machine starts when the button is pressed, and the instrument will stop when the button is pressed again.

Check the following items (Before starting)

- ① Connect the power line to the equipment, after first checking the input voltage supply (220V). This check should be performed when the power is off.
- ② Prepare the designated disinfectants.
- ③ Open the valve of the water supply pipe.
- ④ Remove the debris on the surface of the endoscope where it is to be cleaned and sterilize using a soft wipe with neutral detergent.

### Operating procedure

#### 1) Supply of disinfectant


- ① Select "Supply disinfectant" after power switched on.
- ② Press the "START/STOP" button.
- ③ Inject the disinfectants after opening the lid of the basin.
- ④ Check the "liter(ℓ)" and its volume lamp displayed on the bottom-right of LCD screen.


- ⑤ If the injected volume is equal to 3liters, one lamp will be on. (If it is injected to 7.5liters, two lamps will be on. If it is injected to 10.0liters, three lamps will be on.)

The minimum number of lamps illuminated should be two, for the machine to operate normally. (i.e. At a volume of 7.5 liters or more).

The LCD screen should be displaying the information as below.

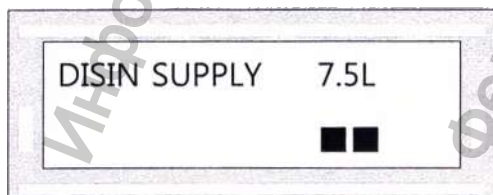
COPY OF ORIGINAL
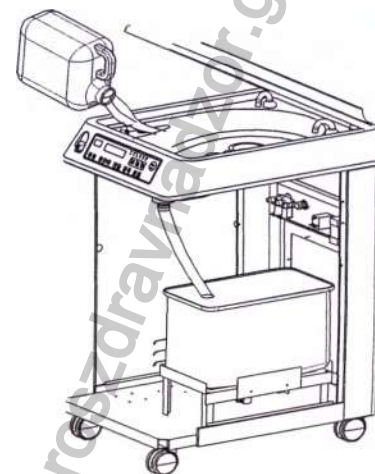
## 2) Cycle Counter (Number of times disinfectant has been used)

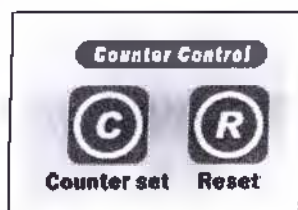- ① Cycle counter allows the user to set or restrict the number of uses of the disinfectant, in accordance with its specification.
- ② Set the number of uses of the disinfectant using the "Up/Down" button, and then pressing the "Counter set" button. Finally press the "Enter" button after setting is complete.
- ③ The Setting counts available are 5 to 150 times maximum.


## 3) Leakage Test (Air check/Under-water check)

► Connection points may vary with different manufacturers and models.


Example ; Olympus endoscope

Please check that the waterproof cap is applied before operating.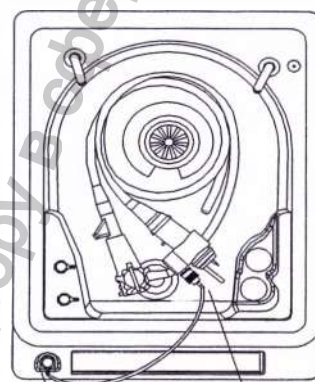


### (1) How to carry out the Air test

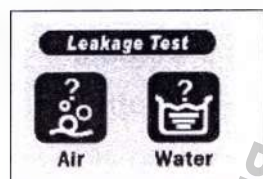- ① Leakage Test should be performed before cleaning/sterilizing.
- ② Place the endoscope ass'y in the basin after applying waterproof cap (the endoscope should be directed to the out-let side)
- ③ Set the air pressure value (220mmHg. typically)
- ④ Press sequentially the "Air check", and "Start" buttons, after first connecting the adaptor which is joined to the leakage test connector (positioned on the left-side of the control panel), to the connector of the endoscope.
- ⑤ If the value of the air pressure setting is displayed, and the value is stable for a while- (E.g. For a period of 2 minutes), it means no leakage is present. (If the pressure setting remains unstable, it means there is a leak somewhere.)

- ⑥ If buzzer rings continuously, perform the underwater test in order to identify the leakage area.

## (2) How to perform the Under-water test

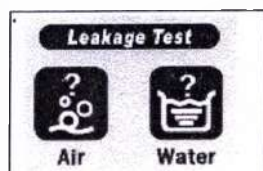- ① The procedures ① to ④ are the same as with the Air check.
- ② Check the leakage with a visual inspection after the endoscope is under water. If there is a leak present, it will be indicated by bubbles rising from the leak source.
- ③ Turn the power off after draining the basin fully.

- ④ Perform the Cleaning/Sterilizing or move them to the circulation chamber, after separating the leakage test adapter which is joined to the endoscope.

An endoscope which shows a leak when tested, should be checked by an SVC engineer.

## (3) Program mode (Cleaning and Sterilization)

► Connection points may vary with different manufacturers and models.


Example of Cleaning/Sterilization ; Olympus endoscope:

- ① Place the endoscope assembly in the basin after first applying the water-proof cap.  
(The endoscope should be directed to the out-let side)
- ② Connect the hose which is joined to the channel connector, to the connector of both suction and air-water channel of the endoscope.
- ③ In mode A, pressing "Enter" causes the processing time to blink. Then change the setting value using the "Up" or "Down" buttons. Finally, press "Enter" to save.


(The recommended conditions are as follows :

Cleaning – 3min Air – 20sec Sterilizing – over 5min)

- ④ Press the "START" button after closing the lid of basin.
- ⑤ Turn the power off, when the voice message is activated, after the processing is completed.
- ⑥ Take it/them out of the basin after separating the hose joined to the endoscope - or move it to the circulation chamber.
- ⑦ Modes "B" and "C" are the same as mode "A".

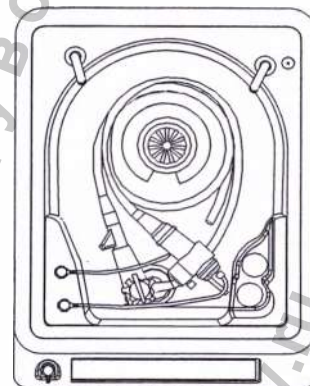COPY OF ORIGINAL
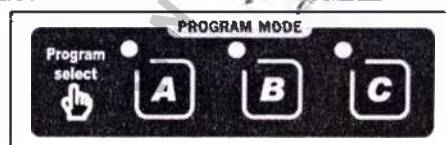
## (5) Air spray/Drain

### ◆ Drain

- ① Select and press "Drain" after the power is switched on.
- ② Press the "START" button.
- ③ It is complete when the water has drained from the basin.  
- The "Drain" function is used when cleaning the basin or removing residual water in the basin.

### ◆ Air

- ① Push the "Air" button after power on. When the value of setting time on the LCD screen is flashing, change the setting time using the "Up" or "Down" button. Press the "Enter" button to save.
- ② Press the "START" button.
- ③ The process is complete when the air setting time has elapsed.  
- The "Air" function is used in case of any remaining water inside the channel of the endoscope.

## (6) Disinfectant drain

- ① Press the "Drain disinfectant" button, after power switched on.
- ② Press the "START" button (The lid of basin should be closed).
- ③ Check on the LCD screen that the "disinfectant" lamps are all off and the disinfectant was all drained successfully.

## 7. During operation (caution)

- ① Do not touch the electrical cord with wet hands.
- ② Only use the machine after electrical earth connected, to prevent the user from electric shock.
- ③ Check the water proof cap is on before using.
- ④ Change the disinfectant if the disinfectant is low. The recommended period of usage is provided by the supplier.
- ⑤ Use the disinfectant setting time when cleaning & sterilizing in accordance with the characteristics and effectiveness of the chemical.
- ⑥ Do not attempt to modify the machine in any way
- ⑦ Only use it after becoming familiar with the operational procedure. Designate an authorized person to operate. It should be handled & operated by the authorized person

COPY OF ORIGINAL

only.

- ⑧ If the machine is faulty, do not attempt to repair it yourself. Contact an SVC engineer or the original supplier, after making a note of the apparent fault.
- ⑨ Do not use it in extremely high or low temperature environments. Be especially careful of frozen or burst pipes in winter.
- ⑩ Cleaness

Basin : Wipe it with a soft wipe, with a mixture of neutral detergent and water. Rinse with fresh water afterwards.

Exterior : The same as with the basin. Use a suitable cleaning product when removing debris or stubborn stains.

## 8. Storage & Maintenance

- ① Keep it clean and Remove any exterior moisture using a dry or dehumidified wipe, for preventing rust later.
- ② Clasp the water supply valve after use.
- ③ For storage, Keep it at room-temperature in a dry environment.
- ④ The connector to the endoscope channel should be separated from the main body before storage.
- ⑤ If not in use for a long time, the stored disinfectant in the basin should be drained, and the electrical cord should be unplugged.

## 9. TECHNICAL SPECIFICATION

- ① Voltage rating : AC 220V,50/60Hz.
- ② Power consumption : 330W
- ③ Cleaning time : 1min. to 90min.  
Sterilization time : 1min. to 90min.  
Air time : 10sec. to 90sec.
- ④ Leakage test
  - Air test : Adjustable from 10 to 250mmHg. Check the leakage by automatic sensor.
  - Underwater test : Adjustable 10 to 250mmHg. Check the leakage by viewing the bubbles in the water.
- ⑤ Disinfectant storage capacity - Maximum : 12l (Typically 8l)
- ⑥ Dimension & Weight
  - Dimension (mm) : 620(L) × 460(W) × 930(H)
  - Weight (kg) : 52.0
- ⑦ Electrical protection category: Class I, Type B Electrical product.

COPY OF ORIGINAL

## 10. Customer Service

### ① Customer's rights (domestic customers in S Korea only)

If a problem arises in use, the Customer has the right to servicing free of charge for one year from the date of purchase. The Service cost will only be charged in the case of mis-handling by the customer, or from some natural disaster (while still within the warranty period).

### ② Customer responsibilities for 'charged' servicing.

- ◆ In the following cases, the service cost will be 'charged' to the customer despite being inside the warranty period.
  - ▶ Breakdowns due to mis-connection or incorrect use of the rated voltage.
  - ▶ Breakdowns due to missing or damaged parts as a result of customer modification.
  - ▶ Breakdowns from natural disasters. , E.g.  
(Thunderbolt falling, Storm, Earth-quake and Fire)
  - ▶ Breakdowns due to mis-handling or re-installation by the customer.

## 11. TROUBLE SHOOTING

Users should perform the following procedure and Never dis-assemble or repair the instrument without the approval of an authorized person.

| Error code | Symptom                      | Checking & Action |                                                                |
|------------|------------------------------|-------------------|----------------------------------------------------------------|
| E01        | No cleaning                  | Check             | . The opening/closing of the basin lid                         |
|            |                              | Act               | . Close the lid of basin                                       |
| E10        | Reduced flow in water supply | Check             | . Valve and Pressure<br>. The filter of water supply system    |
|            |                              | Act               | . Open the valve<br>. Clean the filter of water supply system. |
| E20        | Poor drainage of basin       | Check             | . Filter and drain hose                                        |

|     |                                               |       |                                                                                      |
|-----|-----------------------------------------------|-------|--------------------------------------------------------------------------------------|
|     |                                               | Act   | . Clean the filter of water supply sys.<br>. Straighten out the drain hose           |
| E11 | Poor drainage of basin                        | Check | . The drain lines and level sensors.                                                 |
|     |                                               | Act   | . Clean the filter of the drain lines<br>. Clean the debris out of the level sensors |
| E30 | Lack of disinfectant                          | Check | . Gauge on front panel                                                               |
|     |                                               | Act   | . Re-fill the disinfectant                                                           |
| E32 | No circulation of disinfectant                | Check | . Drain filter/line and level sensors                                                |
|     |                                               | Act   | . Clean the filter of drain lines<br>. Clean the debris out of level sensors         |
| E33 | Excessive disinfectant                        | Check | . Gauge on panel                                                                     |
|     |                                               | Act   | . Drain the disinfectant                                                             |
| E40 | Excessive time to reach correct Air pressure  | Check | . Water proof connection and adaptor                                                 |
|     |                                               | Act   | . Replace the water proof connection and adaptor                                     |
| E41 | Unable to release Air pressure after air test | Check | Contact the manufacturer or sales agent                                              |
|     |                                               | Act   |                                                                                      |
| E21 | Poor water supply                             | Check | . The valve and water pressure<br>. The filter of water supply                       |
|     |                                               | Act   | . Open the valve of water supply<br>. Clean the filter of water supply               |
| E50 | Delay in the supply of disinfectant           | Check | Disinfectant and Basin                                                               |
|     |                                               | Act   | Inject the disinfectant                                                              |
| E60 | Disinfectant not draining                     | Check | Gauge on panel                                                                       |
|     |                                               | Act   | Contact the manufacturer or sales agent                                              |

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----- personally appeared before me and

----- admitted his(her) subscription to the

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This office has been authorized by the  
Minister of Justice, the Republic of  
Korea, to act as Notary Public Since  
7, May. 2009 Under Law No.16.

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